


PREP AND PLACE:

Prep securement area according to facility protocol. Remove release paper on the topside of the device. Orient the Clik-FIX device so the green arrows point towards the insertion site. Place the Clik-FIX device under the wing of the catheter as shown.

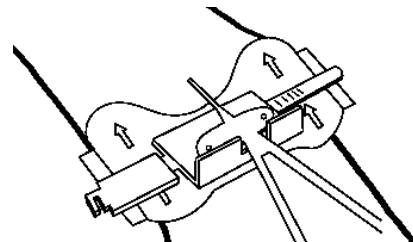


Turn to learn more ↓

1

ALIGN TOP:

Lightly fold the top of the locking mechanism over the wing of the catheter while pushing it slightly forward in the direction of the green arrows. This creates the space required to align the slot on the top onto the strap. The securement strap should be in the vertical position as shown.

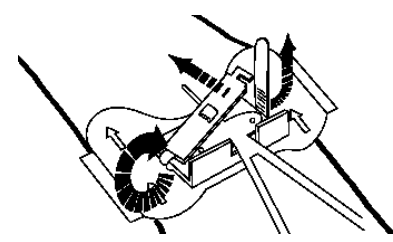


Turn to learn more ↓

2

SLOT ON STRAP:

With the securement strap up in the vertical position, lightly push the top back toward the strap aligning the slot on the top of the locking mechanism up with the securement strap until the strap is in the slot.

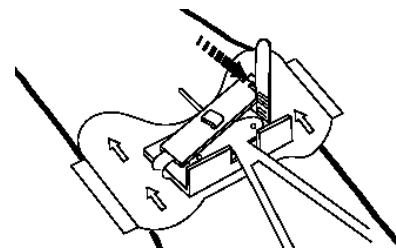


Turn to learn more ↓

3

CLICK TO LOCK:

When the securement strap is in the slot, place index finger under the device and thumb on corner of the top where the strap and the slot mate. Push the top downward until you hear a "click".

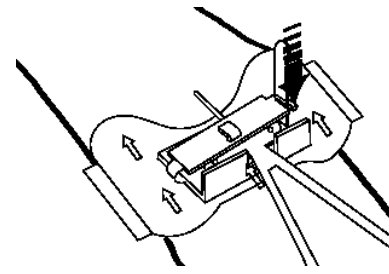


Turn to learn more ↓

4

SECURE STRAP:

After the locking mechanism has been activated, fold the securement strap over and slightly move it forward in the direction of the green arrows. Slide the securement strap under the securement strap holder located on the top of the device.

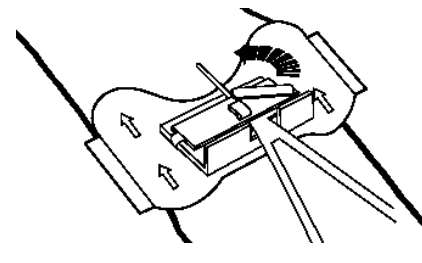


Turn to learn more ↓

5

PEEL AND PRESS:

Remove the center split release paper from the underside of the device one side at a time. Lightly press down on the surface of the Clik-FIX to assure hold to the patient's skin.

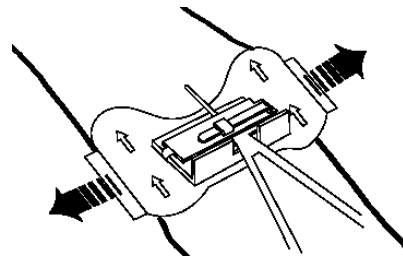

Turn to learn more ↓

6

CLIK-FIX™

PICC/Central Securement



Clik-FIX™ Catheter Securement

-Read carefully before use-

The Clik-FIX device is:

- Single Use Only
- Rx only
- Sterile when packaged in sealed sterile pouch or when included as a component is a sterile kit

Indications: Clik-FIX PICC/Central securement is intended to stabilize compatible PICC and Central Catheters.

Contraindications:

Do not use device on compromised skin i.e., open wounds, burns, lesions, rash, other irritation and/or patient's with known tape or adhesive allergies or hypersensitive skin.

Adverse Reactions: Skin irritation

Note: Adhere Clik-FIX pad after x-ray confirmation of catheter position. Adhesive strip may be applied for added security during dressing changes.

Warning and Pre-Cautions:

- Follow facility protocol when placing and removing the device, observe universal blood and body fluid precautions and infection control guidelines.
- Do not use the Clik-FIX device on patient's who are allergic to adhesives and on non-adherent skin.
- Do not apply alcohol, acetone or other liquid agent directly on the Clik-FIX when in use. These agents can weaken the device.
- During application and removal of the Clik-FIX device take care to minimize manipulation and movement of the catheter.
- Prepare skin surface according to facility protocol. Remove oil and moisture from application area.
- Routinely inspect the Clik-FIX device and catheter position.
- Replace the Clik-FIX device at least every 7 days.


Clik-FIX is a trademark of Starboard Medical, Inc.

Tips for easy use:

TIP: Orient the device so the green arrows on the pad are pointing toward the insertion site.


TIP: Fold the strap up and keep it in the vertical position. Do not bend the strap.

TIP: Slightly move the top of locking mechanism forward in the direction of the green arrows to align slot to strap.

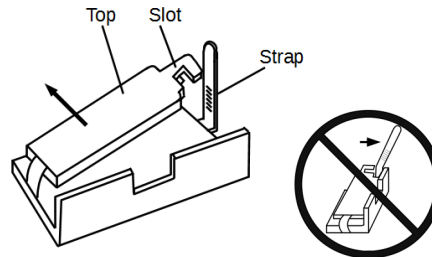

Tips for easy use:

TIP: After aligning the slot on the strap, lightly push the top of the locking mechanism back towards the strap. Push down to "click" secure.

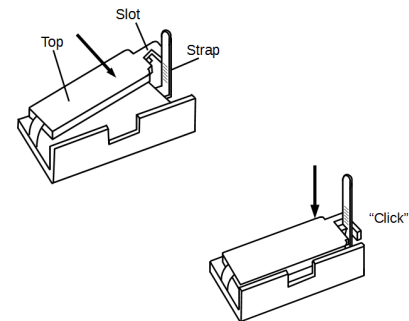

TIP: Follow facility protocol after placement of device, place Chlorhexidine disc, transparent dressing and date and time the dressing.

Removal Technique:

During removal procedure do not stretch the skin. Stabilize the skin while peeling the pad away to reduce skin trauma.

1. Following facility protocol, carefully remove transparent dressing using the stretch technique.
2. Slide the strap forward in the direction of the green arrows to remove it from the strap holder.
3. Lightly move the top of the device away from the strap in the direction of the green arrows to release the strap from the slot and fold to open device.
4. Carefully remove catheter wing from Clik-FIX device.
5. Remove the holder from the skin by peeling the pad gently away from the skin surface from the outside edge.
6. If difficulty is encountered, apply an alcohol prep pad or adhesive removal pad at the interface between the adhesive and skin and peel slowly.

Starboard Medical Inc.
22805 Savi Ranch Pkwy, Suite B, Yorba Linda, CA 92887