



# Clik-FIX™ Luer Lock Securement Device

REF

LLS-1021

## Symbols:

 Read Instructions for Use  Single Patient Use Only  Do not use if package is damaged

 Product does not contain natural rubber latex **STERILE EO** Sterilized with Ethylene Oxide

**R<sub>x</sub>only** Federal (U.S.A) Law restricts this device to be sold by or on the order of a physician.

**Description:** The Starboard Medical Clik-FIX Luer Lock Securement Device (REF: LLS-1021) is designed to provide securement for luer locks, luer lock connections, extension sets, and I.V. associated tubing.

**Indications:** Clik-FIX Luer Lock Securement is intended to stabilize luer lock connections, extension sets, and any I.V. associated tubing and connectors.

**Contraindications:** Do not use device on compromised skin i.e., open wounds, burns, lesions, rash, other irritation and/or patient's with known tape or adhesive allergies or hypersensitive skin.


**Adverse Reactions:** Skin irritation

**Note:** Adhere Clik-FIX Luer Lock Securement to patient to secure luer lock connection, extension sets, and any I.V. associated lines and tubing.

## Warning and Pre-Cautions:

- Follow facility protocol when placing and removing the device, observe universal blood and body fluid precautions and infection control guidelines.
- Do not use the Clik-FIX device on patient's who are allergic to adhesives and on non-adherent skin.
- Do not apply alcohol, acetone or other liquid agent directly on the Clik-FIX when in use. These agents can weaken the device.
- During application and removal of the Clik-FIX device take care to minimize manipulation and movement of the catheters and tubing.
- Prepare skin surface according to facility protocol. Remove oil and moisture from application area.
- Routinely inspect the Clik-FIX device and catheter position.
- Replace the Clik-FIX device at least every 7 days.

## Clik-FIX LLS-1021 Luer Lock Securement Device

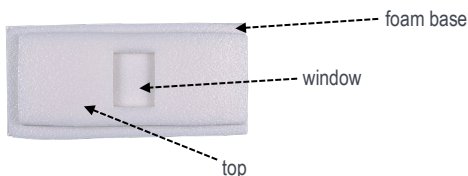

### 1 PREP

Prep and cleanse insertion and securement area according to facility protocol. Allow the area to completely dry.

### 2 PLACE FOAM BASE UNDER LUER LOCK CONNECTOR

Slide the foam base of the luer lock securement device underneath the luer lock connector. This will create a cushioned barrier between the luer lock connector and the patient's skin.

### 3 REMOVE RELEASE PAPER

Remove the center split release paper from the underside of the device one side at a time. Lightly press to down to adhere to skin.

### 4 PEEL, ALIGN AND PRESS

Peel release paper off the back of the top foam with window. Align the window over the luer lock, so the inside edges of the luer lock window surround the perimeter of the luer lock connector. Press to assure hold.



## Removal Technique:

- During removal procedure do not stretch the skin. Stabilize the skin while peeling the pad away to reduce skin trauma.
- Carefully remove the luer lock connector from Clik-FIX device.
- Remove the holder from the skin by peeling the pad gently away from the skin surface from the outside edge.
- If difficulty is encountered, apply an alcohol prep pad or adhesive removal pad at the interface between the adhesive and skin and peel slowly.



Starboard Medical Inc.

18001 Sky Park Circle, Suite E-F, Irvine, CA 92614

Ph. 714.283.3099 Fax 714.283.3033

[www.starboardmedical.com](http://www.starboardmedical.com)

INS-SM-LLS-1021