




Clik-FIX™ Universal Hub/Luer Lock Securement Device

REF

LLS-1024

Symbols:

 Read Instructions for Use  Single Patient Use Only  Do not use if package is damaged

 Product does not contain natural rubber latex **STERILE EO** Sterilized with Ethylene Oxide

R_xonly Federal (U.S.A) Law restricts this device to be sold by or on the order of a physician.

Description: The Starboard Medical Clik-FIX Universal Hub/Luer Lock Securement Device (REF: LLS-1024) is designed to provide securement for catheter hubs, luer lock connections, extension sets, and various compatible medical tubing.

Indications: Clik-FIX Universal Hub/Luer Lock Securement is intended to stabilize catheter hubs, luer lock connections, extension sets, and various compatible medical tubing and connectors..

Contraindications: Do not use device on compromised skin i.e., open wounds, burns, lesions, rash , other irritation and/or patients with known tape or adhesive allergies or hypersensitive skin.


Adverse Reactions: Skin irritation

Note: Adhere Clik-FIX Universal Hub/ Luer Lock Securement to patient to secure catheter hub, luer lock connection, extension sets, or compatible medical tubing and connectors.

Warning and Pre-Cautions:

- Follow facility protocol when placing and removing the device, observe universal blood and body fluid precautions and infection control guidelines.
- Do not use the Clik-FIX device on patients who are allergic to adhesives and on non-adherent skin.
- Do not apply alcohol, acetone or other liquid agent directly on the Clik-FIX when in use. These agents can weaken the device.
- During application and removal of the Clik-FIX device take care to minimize manipulation and movement of the catheters and tubing.
- Prepare skin surface according to facility protocol. Remove oil and moisture from application area.
- Routinely inspect the Clik-FIX device and catheter position.
- Replace the Clik-FIX device at least every 7 days.

Clik-FIX LLS-1024 Luer Lock Securement Device

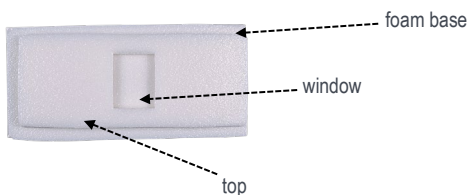

1 PREP

Prep and cleanse insertion and securement area according to facility protocol. Allow the area to completely dry.

2 PLACE FOAM BASE UNDER LUER LOCK CONNECTOR

Slide the foam base of the luer lock securement device underneath the hub or luer lock connector. This will create a cushioned barrier between the luer lock connector and the patient's skin.



3 PEEL, ALIGN AND PRESS

Peel release paper off the back of the top foam with window. Align the window over the hub or luer lock, so the inside edges of the window surround the perimeter of the hub or luer lock connector. Press to assure hold.



④ REMOVE RELEASE LINER

Secure Clik-FIX Universal Hub Securement device to the skin surface by removing the center split release paper from the underside of the device one side at a time. Lightly press to down to adhere to skin.



Removal Technique:

1. During removal procedure do not stretch the skin. Stabilize the skin while peeling the pad away to reduce skin trauma.
2. Carefully remove the luer lock connector from Clik-FIX device.
3. Remove the holder from the skin by peeling the pad gently away from the skin surface from the outside edge.
4. If difficulty is encountered, apply an alcohol prep pad or adhesive removal pad at the interface between the adhesive and skin and peel slowly.



Starboard Medical Inc.

18001 Sky Park Circle, Suite E-F, Irvine, CA 92614

Ph. 714.283.3099 Fax 714.283.3033

www.starboardmedical.com

INS-SM-LLS-1024