





## Clik-FIX™ Neonatal PICC Securement Device

REF

NPS-1006

### Symbols:

-  Read Instructions for Use  
  Single Patient Use Only  
  Do not use if package is damaged  
 Product does not contain natural rubber latex  
 

STERILE	EO
---------	----

 Sterilized with Ethylene Oxide

**R<sub>x</sub>only** Federal (U.S.A) Law restricts this device to be sold by or on the order of a physician.

**Description:** The Starboard Medical Clik-FIX Catheter Securement Device (REF: NPS-1006) is an Engineered Stabilization Device (ESD) designed to provide securement for neonatal peripherally inserted central catheters (PICCs) with wings.

**Indications:** Clik-FIX PICC/Central securement is intended to stabilize compatible PICC and Central Catheters.

**Contraindications:** Do not use device on compromised skin i.e., open wounds, burns, lesions, rash, other irritation and/or patient's with known tape or adhesive allergies or hypersensitive skin.

**Adverse Reactions:** Skin irritation


**Note:** Adhere Clik-FIX pad after x-ray confirmation of catheter position. Adhesive strip may be applied for added security during dressing changes.

### Warning and Pre-Cautions:

- Follow facility protocol when placing and removing the device, observe universal blood and body fluid precautions and infection control guidelines.
- Do not use the Clik-FIX device on patient's who are allergic to adhesives and on non-adherent skin.
- Do not apply alcohol, acetone or other liquid agent directly on the Clik-FIX when in use. These agents can weaken the device.
- During application and removal of the Clik-FIX device take care to minimize manipulation and movement of the catheter.
- Prepare skin surface according to facility protocol. Remove oil and moisture from application area.
- Routinely inspect the Clik-FIX device and catheter position.
- Replace the Clik-FIX device at least every 7 days.


### 1 PREP AND PLACE

Prep securement area according to facility protocol. Remove release paper on the topside of the device to expose the adhesive strip. Orient the Clik-FIX device so the half circle cut out of the foam pad points towards the insertion site. Carefully place the device under the wing of the catheter so the wings are on the adhesive strip at the top of the foam pad. Press lightly to secure.

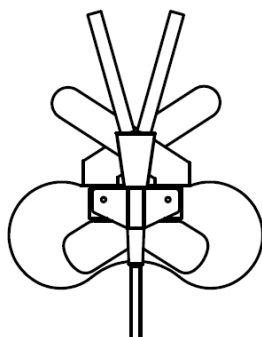


### 2 ALIGN STRAPS

Carefully move the Velcro strap diagonally over the wing of the catheter aligning it to the Velcro hook material on the foam pad.

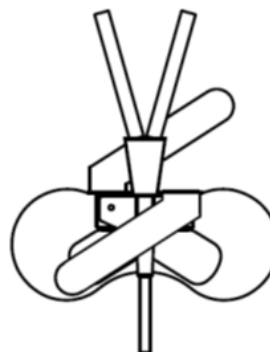

### 3 CROSS OVER AND SECURE

Cross the other strap diagonally over the first one and secure to the Velcro hook material. Verify catheter is secure and Velcro straps have mated appropriately with the Velcro hook material.

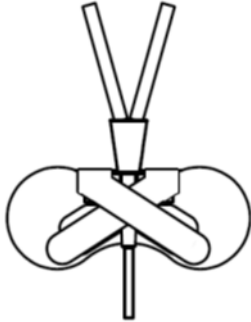

#### TIPS FOR EASY USE:

TIP: Orient the device so half circle cut out in pointing towards the insertion site.

TIP: Place the catheter wing in the device prior to placement of the device on the skin surface.

TIP: After securing the catheter in the Clik-FIX device, follow facility protocol for placement of transparent dressing.

TIP: Always date and time the dressing.

#### Removal Technique:

1. Following facility protocol, carefully remove transparent dressing using the stretch technique. Stabilize the skin while gently removing the dressing.
2. To open, lightly detach the Velcro Straps from the Velcro hook material and carefully remove the catheter wings from the device.
3. Temporarily secure catheter during dressing change with according to facility protocol and clinical best practices.
4. Remove the adhesive pad from the skin by peeling the pad gently away from the skin surface from the outside edge. Follow your facility protocol and clinical best practices for removal of adhesive from skin surface.
5. If difficulty is encountered, apply an alcohol prep pad or adhesive removal pad at the interface between the adhesive and skin to loosen adhesive, continue to carefully remove.
6. Remove device and complete dressing change per facility protocol or if ordered, remove and discard catheter in accordance with your facility protocol.



Starboard Medical Inc.  
18001 Sky Park Circle, Suite E-F, Irvine, CA 92614  
Ph. 714.283.3099 Fax 714.283.3033  
[www.starboardmedical.com](http://www.starboardmedical.com)