


PREP AND PLACE:

Prep securement area according to facility protocol. Remove release paper on the topside of the device. Orient the Clik-FIX device so the green arrows point towards the insertion site. Place the Clik-FIX device under the wing of the catheter as shown.

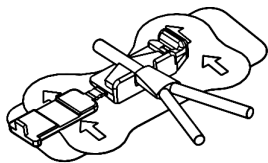


Turn to learn more ↓

1

ALIGN TOP:

Lightly fold the top of the locking mechanism over the wing of the catheter aligning the strap with the slot on base.

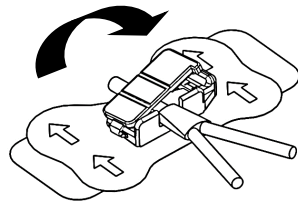


Turn to learn more ↓

2

CLICK TO LOCK:

When the strap is aligned with the slot, place index finger under the device and thumb on the top corner of the device. Push the top downward until you hear a "click".

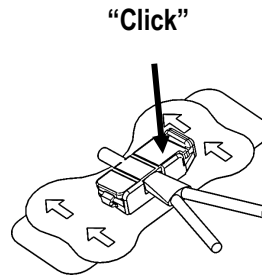


Turn to learn more ↓

3

PEEL AND PRESS:

Remove the center split release paper from the underside of the device one side at a time. Lightly press down on the surface of the Clik-FIX to assure hold to the patient's skin.

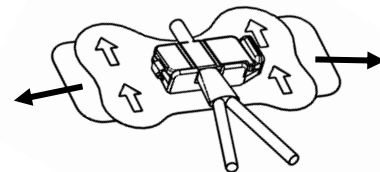


Turn to learn more ↓

4

REMOVAL:

Lightly push the notch on the corner of the top towards the insertion site to open. Do not pull top up, instead gently push top at notch toward insertion site.

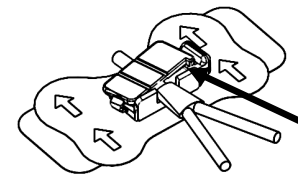

Turn to learn more ↓

5

Removal Technique:

1. Following facility protocol, carefully remove transparent dressing using the stretch technique. Stabilize the skin while gently removing the dressing.
2. To open, lightly push the notch at the top corner of the device towards the insertion site. This will disengage the top and open the device.
3. Carefully remove catheter wing from the Clik-FIX device.
4. Temporarily secure catheter during dressing change with the provided sterile adhesive strip or according to facility protocol.
5. Remove the adhesive pad from the skin by peeling the pad gently away from the skin surface from the outside edge.
6. If difficulty is encountered, apply an alcohol prep pad or adhesive removal pad at the interface between the adhesive and skin to loosen adhesive, continue to carefully remove.
7. Remove device and complete dressing change per facility protocol or if ordered, remove and discard catheter in accordance with your facility protocol.

CLIK-FIX™

PICC/Central Securement



Starboard Medical Inc.
22805 Savi Ranch Pkwy, Suite B, Yorba Linda, CA 92887

Clik-FIX™ Catheter Securement

-Read carefully before use-

The Clik-FIX device is:

- Single Use Only
- Rx only
- Sterile when packaged in sealed sterile pouch or when included as a component is a sterile kit

Indications: Clik-FIX PICC/Central securement is intended to stabilize compatible PICC and Central Catheters.

Contraindications:

Do not use device on compromised skin i.e., open wounds, burns, lesions, rash, other irritation and/or patient's with known tape or adhesive allergies or hypersensitive skin.

Adverse Reactions: Skin irritation

Note: Adhere Clik-FIX pad after x-ray confirmation of catheter position. Adhesive strip may be applied for added security during dressing changes.


Warning and Pre-Cautions:

- Follow facility protocol when placing and removing the device, observe universal blood and body fluid precautions and infection control guidelines.
- Do not use the Clik-FIX device on patient's who are allergic to adhesives and on non-adherent skin.
- Do not apply alcohol, acetone or other liquid agent directly on the Clik-FIX when in use. These agents can weaken the device.
- During application and removal of the Clik-FIX device take care to minimize manipulation and movement of the catheter.
- Prepare skin surface according to facility protocol. Remove oil and moisture from application area.
- Routinely inspect the Clik-FIX device and catheter position.
- Replace the Clik-FIX device at least every 7 days.

Clik-FIX is a trademark of Starboard Medical, Inc.

Tips for easy use:

TIP: Orient the device so the green arrows on the pad are pointing toward the insertion site. Remove release paper on base to expose adhesive and align catheter wings as shown.

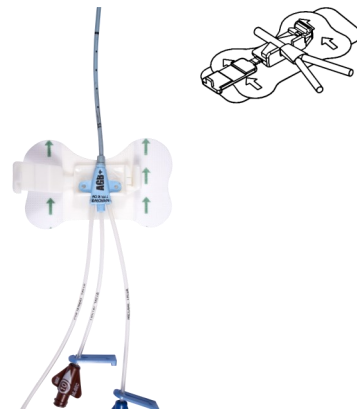

Tips for easy use:

TIP: Follow facility protocol after placement of device, place Chlorhexidine disc, transparent dressing and date and time the dressing.

TIP: Don't try to open the device by pulling or lifting the top up. It is designed not to exert upward pressure that can dislodge the catheter. Instead lightly push the notch on the corner toward the insertion site to open.



Ordering Information:

WCS-1000	PICC/Central Securement (Tricot Pad)	50/bx
WCS-1003	PICC/Central Securement (Foam Pad/Sensitive)	50/bx
PCS-1011	All 'n One Peripheral IV Securement	50/bx
LLS-1021	Luer Lock Securement	50/bx
FCS-1073	Foley Securement	50/bx

For more information:

Call: 714.283.3099

Email: info@starboardmedical.com

www.starboardmedical.com

