

Mask2Shield™

Protective Shield

Personal protective equipment is crucial, now more than ever, to prevent the spread of infectious disease like the novel coronavirus. Mask2Shield is a one time use, lightweight eyeshield designed to attach to any surgical or respirator mask. Mask2Shield was designed to protect the face from hazardous droplets and sprays while also being comfortable to wear for extended periods of time. While wearing Mask2Shield, there will be no additional pressure behind the wearer's ears. The mask may be worn with or without the included elastic string. Either way, Mask2Shield will provide adequate facial protection.

Features	Benefits
Effortless PPE	Converts a mask to a shield in seconds
Disposable	Reduces risk of cross contamination
Full Frontal and Lateral Coverage	Helps reduce the risk of eye splash contamination
Optically Clear	Good visibility
Low Fog Lense	Reduces blurring
Versatile Fit	Can be worn in tandem with other PPE, hair coverings, and prescription glasses
Includes Optional Headstrap	No additional pressure behind the wearer's ear
3 Strap Positions	Accommodates various head/hair sizes
Comfortable and Lightweight	Reduces discomfort associated with heavy shields



It's as easy as 1, 2, 3.


1 Peel the easy-release backing off of the adhesive tabs.



2 Attach to the top of a standard surgical or N95 mask and press.



3 Secure the barbed ends of the strap in the holes that fit you best.

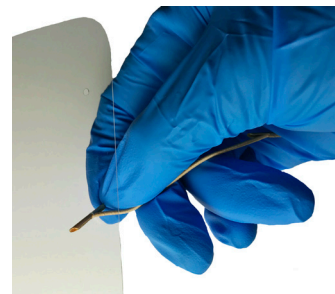

18001 SkyPark Cir, Suite E-F, Irvine, CA 92614
Phone: 714.283.3099 • Fax: 714.283.3033 • starboardmedical.com

