


Clik-FIX™ Neonatal PICC Securement Device for Neonatal PICCs with Silicone Wings

REF NPS-1006

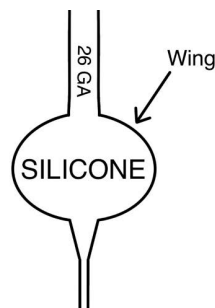

Symbols:

- Read Instructions for Use
- Single Patient Use Only
- Do not use if package is damaged
- Product does not contain natural rubber latex
- STERILE EO** Sterilized with Ethylene Oxide

R_xonly Federal (U.S.A) Law restricts this device to be sold by or on the order of a physician.

Description: The Starboard Medical Clik-FIX Catheter Securement Device (REF: NPS-1006) is an Engineered Stabilization Device (ESD) designed to provide securement for neonatal peripherally inserted central catheters (PICCs) with silicone wings.

Indications: Clik-FIX PICC/Central securement is intended to stabilize compatible PICC and Central Catheters with silicone wings.

Contraindications: Do not use device on compromised skin i.e., open wounds, burns, lesions, rash, other irritation and/or patient's with known tape or adhesive allergies or hypersensitive skin. Do not use the device with catheters that do not have silicone wings.

Adverse Reactions: Skin irritation


Note: Adhere Clik-FIX pad after confirmation of catheter position.

Warnings and Precautions:

- Follow facility protocol when placing and removing the device, observe universal blood and body fluid precautions and infection control guidelines.
- Do not use the Clik-FIX device on patients who are allergic to adhesives and on non-adherent skin.
- Do not apply alcohol, acetone or other liquid agent directly on the Clik-FIX when in use. These agents can weaken the device.
- During application and removal of the Clik-FIX device take care to minimize manipulation and movement of the catheter.
- Prepare skin surface according to facility protocol. Remove oil and moisture from application area.
- Routinely inspect the Clik-FIX device and catheter position.
- Replace the Clik-FIX device at least every 7 days.


1 PREP AND PLACE

Prep securement area according to facility protocol. Remove release paper on the topside of the device to expose the oval adhesive patch. Orient the Clik-FIX device so the half circle cut out of the foam pad points towards the insertion site. Carefully place the device under the silicone wing of the catheter so the wing is on the oval adhesive patch on top of the device. Be sure to align the back of the wing with the edge of the device. Press lightly to secure.

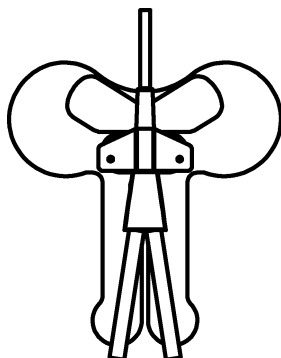


2 ALIGN STRAPS

Carefully move the hook-and-loop strap diagonally over the wing of the catheter aligning it to the hook-and-loop material on the base of the device. The strap should be snug against the back of the catheter wing.

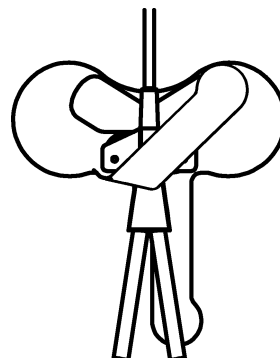


3 CROSS OVER AND SECURE

Cross the other strap diagonally over the first one and secure to the hook-and-loop base. Verify catheter is secure and hook-and-loop straps have mated appropriately with the base of the device

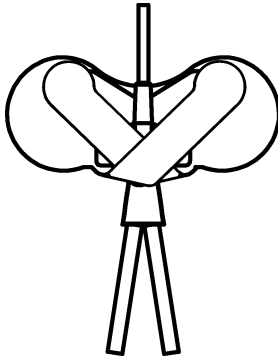

4 APPLY TO SKIN

Secure the Clik-FIX device to the skin surface by removing the release papers from the underside of the device one side at a time. Lightly press to down to adhere to skin. Follow facility protocol for use of transparent dressing.

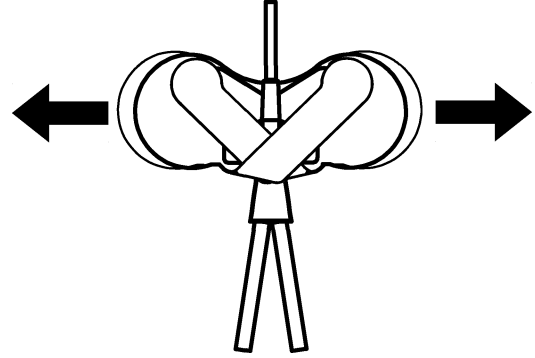

Removal Technique:

1. Following facility protocol, carefully remove transparent dressing. Stabilize the skin while gently removing the dressing.
2. To open, lightly detach the hook-and-loop straps from the base of the device and carefully remove the catheter wings from the device by "rolling" the wings off of the device, starting with one side of the wings. If difficulty is encountered when removing the wings from the base of the device, use gauze dampened with NS or sterile water, or your facility's protocol for adhesive removal for neonates.
3. Temporarily secure catheter during dressing change according to facility protocol and clinical best practices.
4. Remove the adhesive pad from the skin by peeling the pad gently away from the skin surface from the outside edge. Follow your facility protocol and clinical best practices for removal of an adhesive pad from skin surface. If difficulty is encountered when removing the device, use gauze dampened with NS or sterile water, or your facility's protocol for adhesive removal for neonates.
5. Remove device and complete dressing change per facility protocol or if ordered, remove and discard catheter in accordance with your facility protocol.

TIPS FOR EASY USE:

- TIP: Orient the device so half circle cut out is pointing towards the insertion site.
- TIP: After securing the catheter in the Clik-FIX device, follow facility protocol for placement of transparent dressing.
- TIP: Always date and time the dressing.



Starboard Medical Inc.
18001 Sky Park Circle, Suite E-F, Irvine, CA 92614
Ph. 714.283.3099 Fax 714.283.3033
www.starboardmedical.com